

pH-Fix 3.1–8.3

en

Test strips for measuring pH values in the range of 3.1–8.3



IVD IVD for use by
qualified personnel only

i Please refer to
instructions for use

REF 921 35

Pack content:

100 test strips
1 reference color scale pH 3.1–8.3

Application:

pH-Fix test strips pH 3.1–8.3 are designed for measuring the pH value of saliva. To be used only by qualified personnel. Please refer to a qualified professional for an interpretation of the test results. Checking the test strips: In order to verify that the test strips work correctly, please check them by using artificial saliva which is available on the market.

Instructions for measuring the pH value of saliva:

1. Take a saliva sample and dip the test strip into the sample for approximately 5 seconds.
2. Dab the strip along the edge on a paper towel to remove excess liquid.
3. Immediately compare the test fields with the color scale. Note down the value which matches closest with the colored test fields. The color evaluation must be performed immediately, as the color fields will continue to change in intensity (accuracy of reading: ± 0.2 pH).

Storage:

Avoid exposing the strips to sunlight and/or moisture. Keep container cool and dry (store below +30 °C).

Disposal:

Used test strips may be disposed of as domestic waste.



Declaration of conformity (Product complies with the In Vitro Diagnostics Directive 98/79/EC of the European Union)	In Vitro Diagnostic Product
Use by (not to be used after the use-by-date displayed on the packaging)	Batch identification
Item number (for reordering purposes)	Not suitable for reuse. Dispose of after use.
Storage temperature (4–30 °C) – Store at room temperature, and do not expose to direct sunlight.	Please refer to instructions for use! Prior to use, please read these instructions carefully.